



ΘΕΟΔΩΡΟΠΟΥΛΟΣ Α. ΙΩΑΝΝΗΣ
Sanaflex Anatomic
Κυπρου 93, Athens, 164 51



A-111
Υποστήριξη Κοιλίας / Abdominal Support

ΜΕΓΕΘΟΛΟΓΙΟ

One Size
(S - XL)

GR | Ελληνικά

A-111, Υποστήριξη Κοιλίας

ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ

ΕΝΔΕΙΞΕΙΣ

Ιδανική για την υποστήριξη των κοιλιακών μυών, για ανακούφιση της σπονδυλικής στήλης, μετεγχειρητικά, έπειτα από λιποαναρρόφηση, υστερεκτομή. Μπορεί να χρησιμοποιηθεί και ως ζώνη σύσφιξης.

ΤΟΠΟΘΕΤΗΣΗ

Τυλίξτε τη ζώνη γύρω από την μέση σας, βεβαιωθείτε ότι είναι τοποθετημένη στο κέντρο της πλάτης σας και ασφαλίστε με τα Velcro

Εκτός από τη μέτρηση, είναι απαραίτητη η δοκιμή για τη σωστή εφαρμογή και την κατανόηση της ορθής τοποθέτησης του προϊόντος. Προσαρμόστε τη ζώνη και την ένταση της ασκούμενης πίεσης κατά βούληση, ανάλογα με την ένταση του πόνου και τη δραστηριότητα.

ΑΝΤΕΝΔΕΙΞΕΙΣ

Μη χρησιμοποιείτε το προϊόν αν είστε αλλεργικός/η σε κάποιο από τα συστατικά του. Σε περίπτωση εμφάνισης ερεθισμού, διακόψτε τη χρήση του προϊόντος και συμβουλευτείτε το γιατρό σας.

ΣΥΝΤΗΡΗΣΗ

Πλύσιμο στο χέρι σε νερό στους 30°C με ήπιο σαπούνι ή απορρυπαντικό. Ξεπλύνετε καλά, μην στίβετε, αφήστε να στεγνώσει, μη σιδερώνετε.

Σύνθεση

80% Neoprene, 20% Nylon
Spacer Fabric: 45% Polyester, 55% Spandex



EN | English

A-111, Abdominal Support

INSTRUCTION MANUAL

INDICATIONS

Abdominal support for the weak and strained muscles. Comfortable to wear thanks to elastic and breathable material. Ideal to support abdominal musculature, to relieve pressure on the spinal column, post-surgical management. Can be used as a waist trimmer.

FIT

Wrap the belt around your abdomen, make sure it is positioned on the centre of your back and secure the Velcro fasteners

In addition to measuring the waist, the belt should be tried on to ensure a good fit and to understand how to put it on properly. Tighten the belt based on the intensity of the pain and type of activity.

CAUTIONS

Do not use the product if you are allergic to one of its components. In case of any irritation occurrence, discontinue the use of the product and seek medical advice.

CARE AND MAINTENANCE

Hand wash in warm soapy water (maximum 30°C). Dry flat, away from sources of excessive heat (radiator, tumble dryer).

Material Composition

80% Neoprene, 20% Nylon
Spacer Fabric: 45% Polyester, 55% Spandex

